Wellpoint New Jersey

P. O. Box 62509
Virginia Beach, VA 24466-2509

<Date>

<Parent or Guardian of>

<Member Name>

<Member Address>

<Member City, State ZIP>

Dear Member or Parent/Guardian of <Member Name>:

We have important Medical Device Correction information about your prescription for Omnipod® 5 Pods.

Our records show that you may have recently filled a prescription for this product.

On March 12, 2026, Insulet Corporation issued a Medical Device Correction of certain lot numbers of Omnipod® 5 Pods.

Insulet Corporation issued this Medical Device Correction after identifying a manufacturing issue through its ongoing product monitoring. According to Insulet Corporation, certain Pods from specific lots may have a small tear in the internal tubing that delivers insulin. If this occurs, insulin may leak inside the Pod instead of being fully infused into the body as intended.

Insulet Corporation has advised that if insulin is not delivered properly, users may experience high blood glucose levels due to under-delivery of insulin.

Product: Omnipod® 5 Pods

NDC: 08508-3000-21

Lot number: PH1U02252541, PH1U03282511, PH1U03282522,
PH1U03312511, PH1U03312521, PH1U04012511,
PH1U04012521, PH1U05052511, PH1U08162531,
PH1U08182531, PH1U08182541, PH1U09242511,
PH1U09242521, PH1U09242531, PH1U09252521,

wellpoint.com/nj/medicaid

1090271NJMESWLP 04/26

OMHC# 078-25-59

105-CNMCD26024WLPNJ 032326

PH1U09252531, PH1U10152541, PH1U10162531,
PH1U10162541, PH1U10172531, PH1U10172541,
PH1U10182531, PH1U10182541, PH1U10202511,
PH1U10202521, PH1U10202531, PH1U10202541,
PH1U10212531, PH1U10212541, PH1U10222531,
PH1U10222541, PH1U10232531, PH1U10232541,
PH1U10242521, PH1U10242531, PH1U10242541

Talk to your doctor before you stop taking any medication. Stopping a drug without a replacement could have health risks. Your doctor can talk to you about other options that are right for you.

What you need to do next:

- Check your prescriptions to see if you have Omnipod® 5 Pods.
- To see if you have affected product, check the lot number. The lot number can be found on the Pod tray lid, on the flat bottom of the Pod itself, and also on the side of the 5-Pack Pods box. All Pods in the same box will have the same lot number.
- You may also visit omnipod.com/check-pods and enter all digits after PH1U from your Pod lot number.
- Insulet Corporation has advised that consumers immediately discontinue use of the product affected by this Urgent Medical Device Correction. Consumers with questions regarding this Medical Device Correction can contact Insulet Product Support team at 1-800-641-2049 (available 24/7) or use the live agent chat at omnipod.com/current-podders.
- Talk with your doctor about other products to use instead.
- Find more information about this Medical Device Correction at FDA.gov/medwatch.

We're here to help

If you have questions about the Medical Device Correction, please call Insulet Corporation, the manufacturer, at 1-800-641-2049 (available 24/7) or use the live agent chat at omnipod.com/current-podders. You can contact the FDA toll-free at **888-INFO-FDA (888-463-6332)** or visit fda.gov for more information.

You can also call your doctor or pharmacist if you have questions about the product or about this letter. If you have questions about your prescription drug benefits, please call Pharmacy Member Services at the phone number on your member ID card (TTY users can call **711**).

wellpoint.com/nj/medicaid

1090271NJMESWLP 04/26

OMHC# 078-25-59

105-CNMCD26024WLPNJ 032326

Sincerely,

Your Wellpoint service team

Enclosures: Language assistance sheet
Nondiscrimination notice

This information is provided for educational purposes only and should not be considered medical advice. Talk to your physician for specific treatment options and recommendations best suited for you and before beginning any lifestyle program.

[wellpoint.com/nj/medicaid](https://www.wellpoint.com/nj/medicaid)

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